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The screenshot shows the user profile page of e-okusuran. The top navigation bar includes 'eお薬さん', 'ホーム' (Home), '服薬情報' (Medication Information), '機器情報' (Device Information), 'お知らせ' (Notice), and a user profile icon labeled 'ヘルプ' (Help) and '衛材花子' (Eisai Hanako), which is circled in red. The main content area has a '通信中' (Communicating) status bar with a question mark icon, a name '氏名: 衛材花子', and a 'メッセージ' (Message) section. Below this is a '服薬情報' (Medication Information) table with columns for '設定時間' (Setting Time), 'あさ' (Morning), 'ひる' (Afternoon), 'よる' (Evening), and 'ねる前' (Before Sleep). The table shows '未設定' (Not Set) for all times. On the right, there are two notice boxes: '管理者からのお知らせ' (Notice from Administrator) and 'システム管理者からのお知らせ' (Notice from System Administrator), both indicating no notices are present.

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The screenshot shows the 'eお薬さん' interface. At the top, there are navigation tabs: ホーム (Home), 服薬情報 (Medication Information), 機器情報 (Device Information), and お知らせ (Notice). The '服薬情報' tab is selected. Below the tabs, there is a header bar with the user's name '衛材花子' and a help icon. The main content area is divided into two sections. The left section, titled '通信中' (Communicating), shows a profile icon with a question mark, the name '衛材花子', and a message input field. The '送信' (Send) button is circled in red. The right section, titled '管理者からのお知らせ' (Notice from Administrator), shows a list of notices, including one dated 11月01日 about the release of the eお薬さん cloud service.

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The screenshot shows the 'eお薬さん' interface after the message has been sent. A yellow message bar at the top indicates 'メッセージを受け付けました。' (Message received). The '送信' (Send) button is now disabled. The right section, titled '管理者からのお知らせ' (Notice from Administrator), shows a list of notices, including one dated 11月01日 about the release of the eお薬さん cloud service.

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The screenshot shows the 'eお薬さん' interface with the '服薬情報' (Medication Information) tab selected. The '送信' (Send) button is circled in red. The right section, titled '管理者からのお知らせ' (Notice from Administrator), shows a list of notices, including one dated 11月01日 about the release of the eお薬さん cloud service.

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The screenshot shows the eお薬さん homepage. The navigation bar at the top includes 'ホーム' (Home), '服薬情報' (Medication Information), '機器情報' (Device Information), and 'お知らせ' (Notice). The '機器情報' item is circled in red. Below the navigation bar, there is a '通信中' (Communicating) status section with a user profile for '衛材花子' (Eizai Hanako). To the right, there are two notification boxes: '管理者からのお知らせ' (Notice from Administrator) and 'システム管理者からのお知らせ' (Notice from System Administrator). At the bottom, there is a '服薬情報' (Medication Information) section with a table showing medication times: 'あさ' (Morning), 'ひる' (Afternoon), 'よる' (Evening), and 'ねる前' (Before Bed).

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The screenshot shows the '機器情報' (Device Information) page. The page title is '機器情報'. Below the title, there is a table with the following information:

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Below the table, there is a 'ログ' (Log) section. It contains a table with columns '日時' (Date/Time), 'コード' (Code), and '内容' (Content). The table has one row of data:

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Navigation buttons '前へ' (Previous) and '次へ' (Next) are located to the right of the log table.

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This screenshot is identical to the one above, showing the eお薬さん homepage with the '機器情報' (Device Information) menu item circled in red. It displays the user profile, notification boxes, and the medication information table.

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© 2015 - 2017 by And-Fun Ltd. All rights reserved.
© 2015 - 2016 by And-Fun Ltd. All rights reserved.